
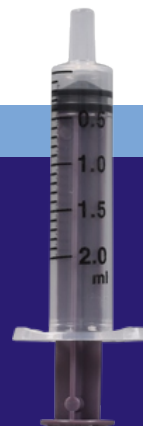
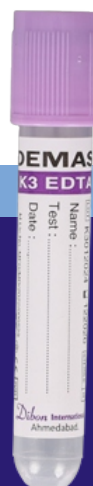


**prompt**<sup>®</sup>  
The pursuit of purity

# Pregnancy detection in **just 28 days** **post AI**

**prompt**<sup>®</sup>  
**BOVEASY**  
bovine pregnancy rapid test kit

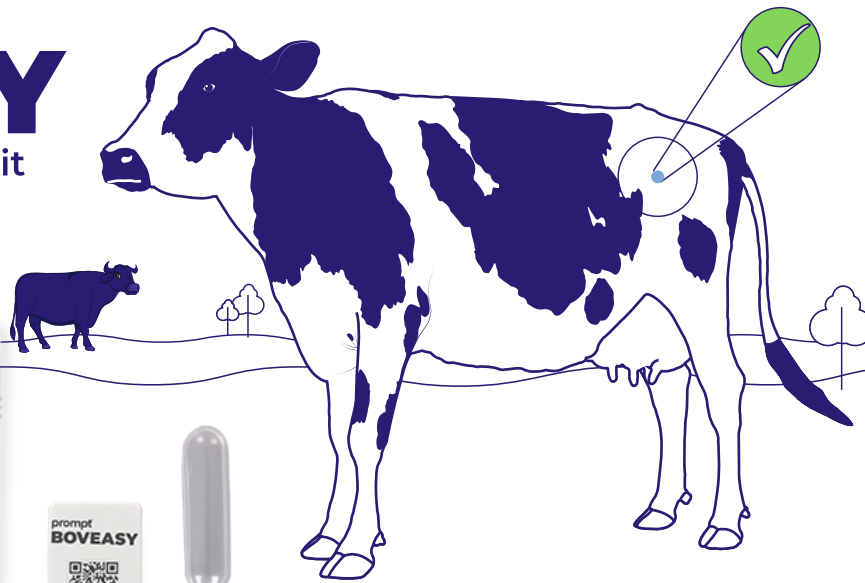

# Why prompt **BOVEASY**?

bovine pregnancy rapid test kit



Detect pregnancy in just 28 days after AI

Non-invasive test method makes it safe for embryos and imported semen

Reduces calving interval, saving up to Rs. 6,000/- per animal


98% accuracy rate as it detects pregnancy-associated glycoprotein



## 3 simple steps


### 01 Sample Collection

Collect 2-3 ml of blood from the tail vein, jugular vein, or ear vein of the animal or after the 28<sup>th</sup> day of artificial insemination.

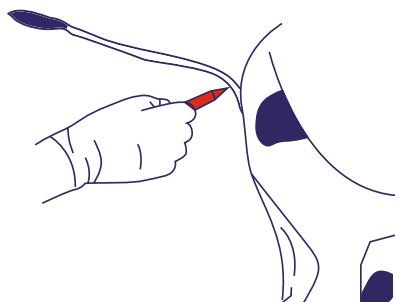


### 02 Sample Preparation

Add 2-3 drops of blood into the sample well of the test strip.

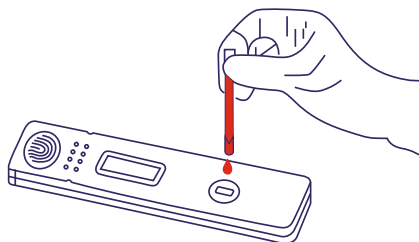

### 03 Adding Diluent

Add 2 drops of diluent in the sample well. Get results in just 20 minutes. For accurate interpretation, make sure you read the results within 10 minutes.

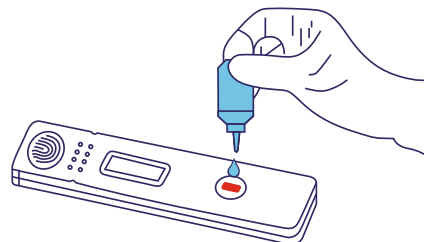


## Result interpretation


Quick Test: for multiple samples

°C: Incubate at room temperature (18-28°C) for 20 min.

Note: Please interpret the result within 10 min. after the incubation.

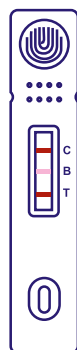


PREGNANT

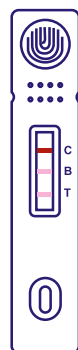

SUSPICIOUS



NON PREGNANT

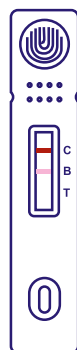

INVALID

Scan the QR Code to know more:



**Prompt Equipments Pvt. Ltd.**

7th Floor, Shaligram Corporates, C. J. Road, Iskon - Ambli Road, Ahmedabad Gujarat - 380058 (India)

info@promptdairytech.com | www.promptdairytech.com

Ph: 02717 45 1111/6111 | Follow us on    