
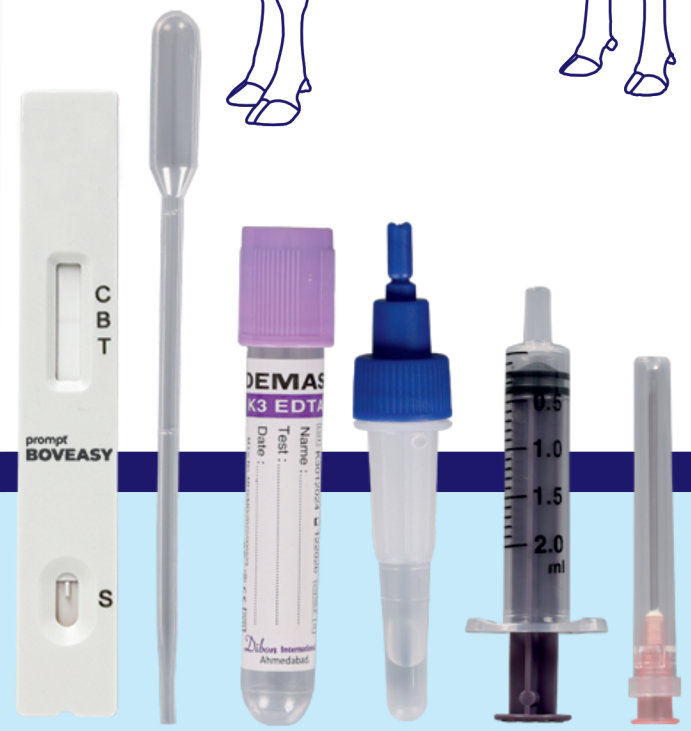


prompt[®]
The pursuit of purity

Early pregnancy detection in **just 28 days** **post AI**

prompt[®]
BOVEASY
bovine pregnancy rapid test kit

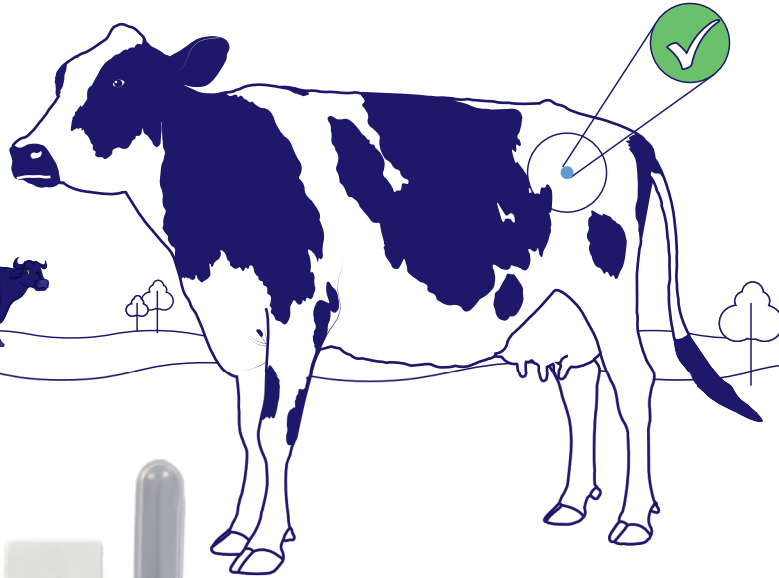

Why prompt BOVEASY?

bovine pregnancy rapid test kit



Detect pregnancy in just 28 days after AI

Non-invasive test method makes it safe for embryos and imported semen

Reduces calving interval, saving up to Rs. 6,000/* per animal

98% accuracy rate as it detects pregnancy-associated glycoprotein



3 simple


01 Sample Collection

Collect 2-3 ml of blood from the tail/ jugular/ ear vein of the bovine or after the 28th day of artificial insemination.




02 Sample Preparation

Add 2-3 drops of blood into the sample well of the test strip.

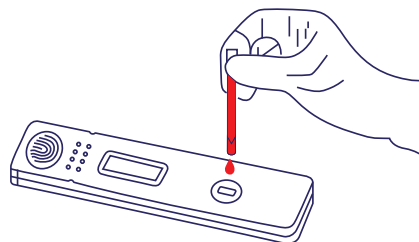


03 Adding Diluent

Add 2 drops of diluent in the sample well. Get results in just 20 minutes. For accurate interpretation, read the results within 10 minutes.

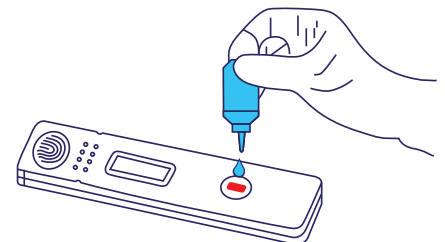


Get results in 10 minutes

The test kit has three lines, C is the control line, B is the base line and T is the test line. If the test line T appears darker red/pink than the intermediate baseline, it proves that the animal is pregnant.

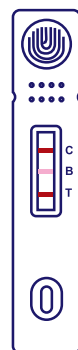



PREGNANT

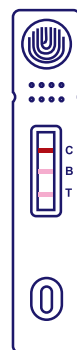

SUSPICIOUS



NON PREGNANT

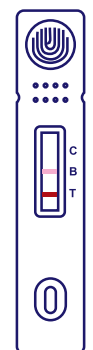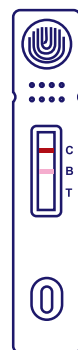

INVALID

Prompt Equipments Pvt. Ltd.

7th Floor, Shaligram Corporates, C. J. Road, Iskon - Ambli Road, Ahmedabad Gujarat - 380058 (India)
 info@promptdairytech.com | www.promptdairytech.com
 Ph: 02717 45 1111/6111 | Follow us on

Scan the QR Code to know more:

